



## ASSEMBLY & PACKING INSTRUCTIONS FOR THE MULTICHAIR 4000tx

### NOTICE:


Please see section 3: LIMITED WARRANTY. If warranty is not acceptable, return the chair for a full refund. Contact Nuprodx, Inc. for shipping instructions.

### 1 ASSEMBLY INSTRUCTIONS:


**A** Unpack all parts from the shipping carton. Remove all tape and/or packing from the chair. Keep shipping materials for future use.

**B** Unfold the seat frame, LOCK BOTH OF THE SLIDE BOLTS, and position the frame upside down.

multiCHAIR 4000tx and carrying case

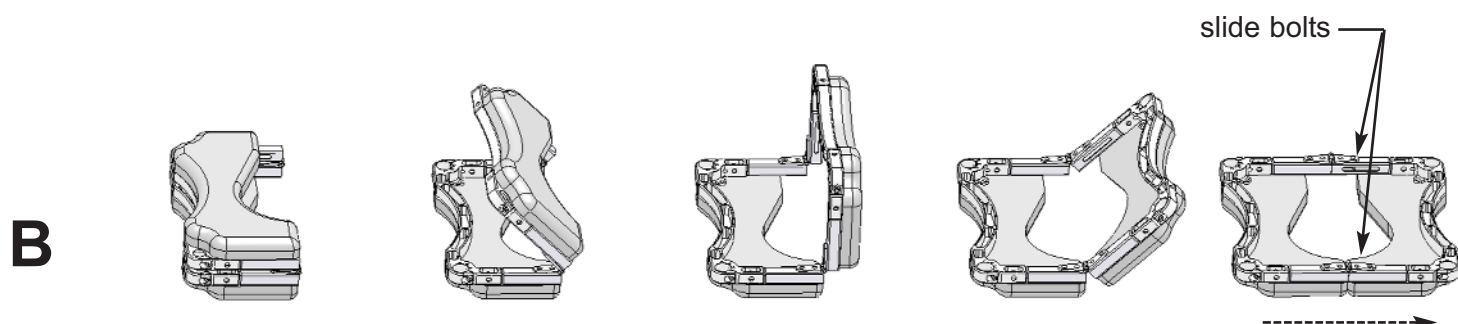

**B**

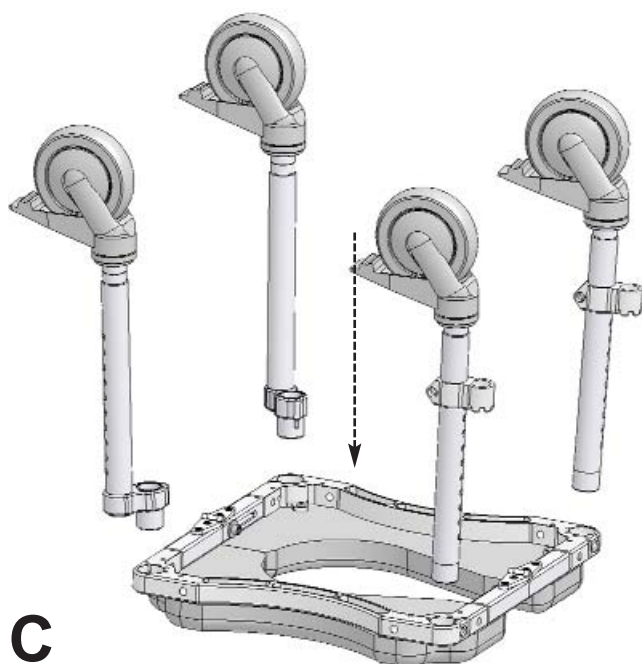


**C**


**C** Position the seat frame upside down and install the four legs. Note that the top of each leg has a machined portion approximately one inch wide. The back two legs have a 1" offset or elbow attached to the end of each leg.

**D** 1) Position the back two legs so that the offset points out at a 45-degree angle (note the small pin that mates with holes in the frame). The top end of each leg should fit easily into the socket of the frame. Make sure that the legs slide "home" into the sockets. Use the included 1/4" Allen wrench to tighten the bolts located at the corners of the frame. For ease of future leg height adjustment, position each leg in the socket so that the adjustment holes face outward.


2) Use the included 1/4" Allen wrench to tighten the bolts located at the corners of the frame. For ease of future leg height adjustment, position the front legs in the socket so that the adjustment holes face outward.

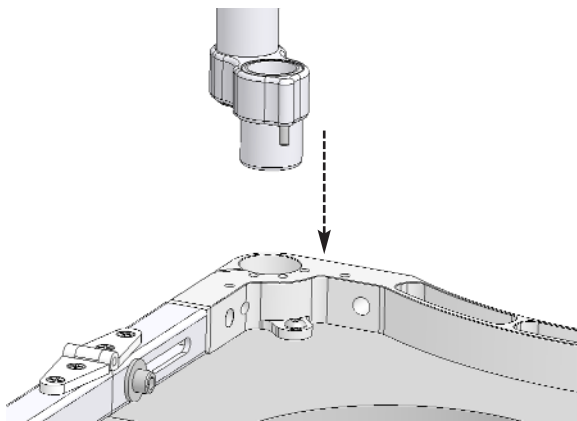

**D**

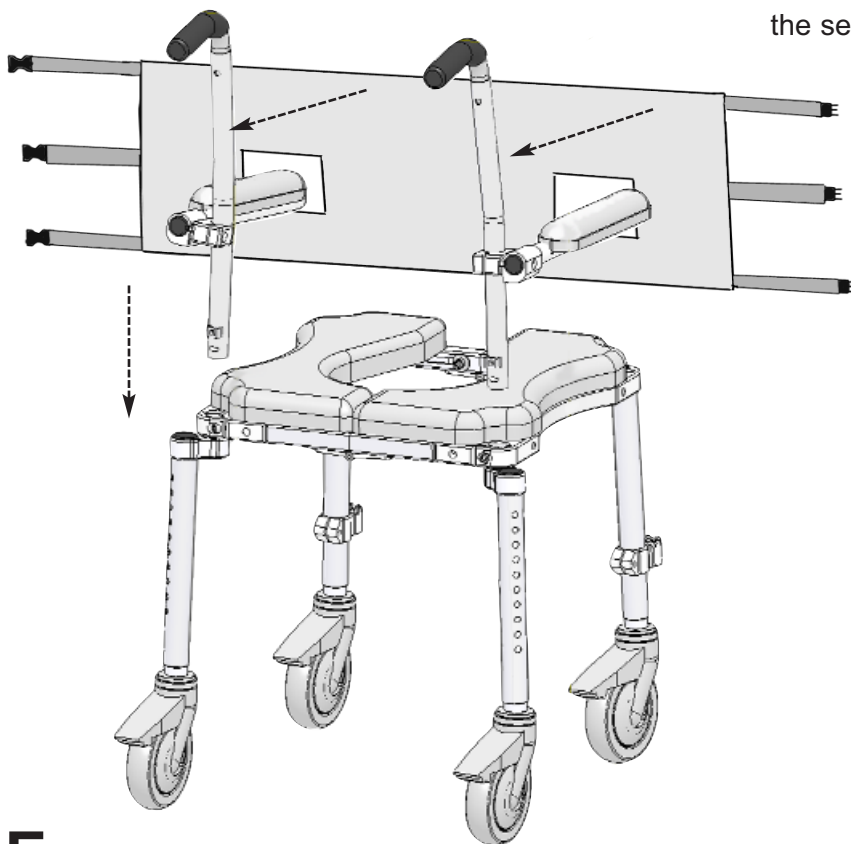

**E**

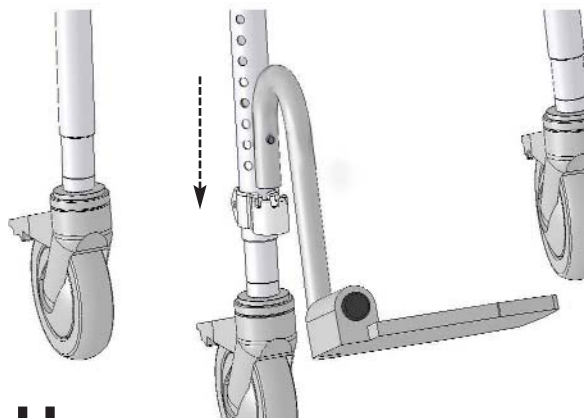

**H**


**E** 1) Turn the chair right side up and install the backrest assembly (which includes two posts and a cloth back). Each of the posts has an internal clamp that pushes on the inside of the back legs.

2) Note that the aluminum saddle washers, on the backrest posts) have a semicircle machined to it which mates with matching points on the seat frame. This feature prevents rotation of the backrest posts.

3) Attach the cloth back by threading it through the cantilever arms, wrapping it around the backrest posts, and pressing it against the velcro. Back angle adjustments are possible by changing the length of the straps.

**F** This image is a close up of the seat frame showing the semi-circles.

**G** This image is a close up of the saddle washer showing the semi-circle.

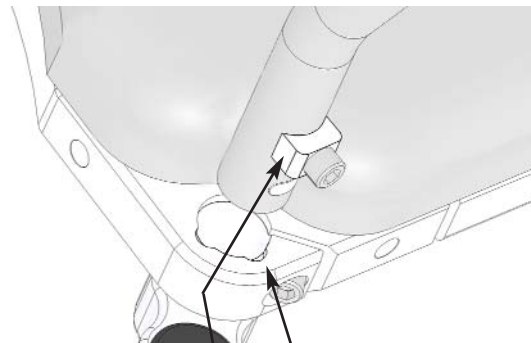


**F**

saddle washer

semi-circle

**G**

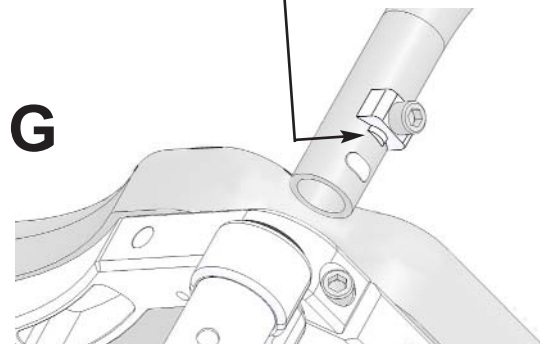


**H** Install the swing-away footrests:

1) Footrest clamps are installed on the front legs at the factory.

2) Loosen the pinch bolt and slide up/down as required by the users leg length

3) Slide the gooseneck tubing/foot paddle into the clamp

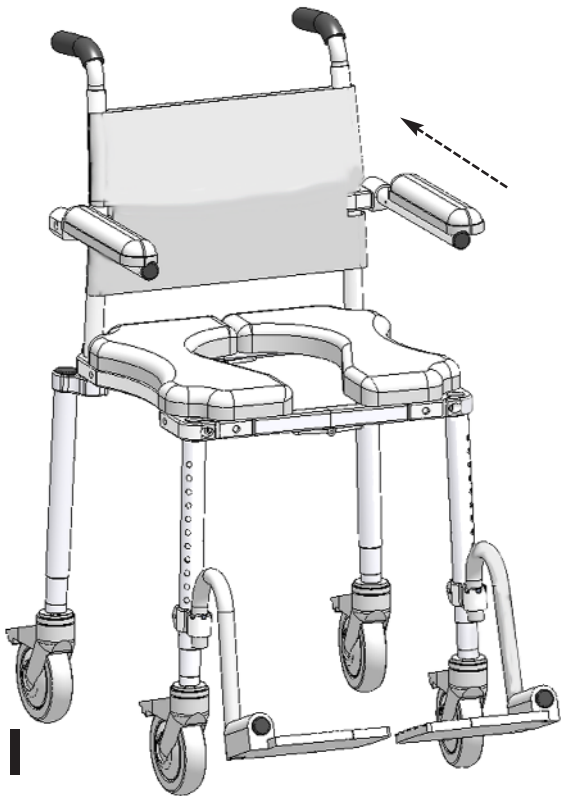

1) Cantilever side arms: adjust up/down/in/out by slightly loosening the bolts on the inside face of the clamp attached to the backrest posts - make the adjustment and retighten.

2) Unlocking the arms (for transfer access) by pressing rearward on the front edge and pulling up.

## 2 OPERATING INSTRUCTIONS:

**A Leg Adjustment:** When you adjust the legs, make sure that the brass buttons are fully seated in the legs. Note: if you push too hard the button could become stuck inside the inner leg; pop out the black end cap to fix this.

**B Lock the brakes on the casters by pushing down on the textured rear-most part of the caster brake levers. Release the brakes on the casters by pushing down on the front textured part of the caster brake levers.**

**C Seat cushion:** The cushion is quality closed-cell foam, designed to provide comfort and protection. Make sure that you don't pack anything on top of it - like the back assembly, arm rests, or any tools. Because it doesn't have air bubbles, the foam will not spring back after being compressed for a long time: for example on an airplane trip. Any dents in the foam will not spring back and are permanent.

**D Periodically check the tightness of all bolts and apply light oil to the threads of the bolts.**

## 3 LIMITED WARRANTY

**A Please read this warranty before operating or using your Multichair 4000tx. By operating or using the chair, you agree to the terms of this warranty. If you do not agree, return the chair to Nuprodx, Inc. and you will receive a full refund of the price you paid.**

**B WARRANTY:** Nuprodx, Inc. warrants this product against defects in material and workmanship as follows: There is a 30-day conditional money-back warranty. During that period the chair can be returned for a full refund minus a 20% restocking charge. After that period, there is a three-year limited warranty for all parts except cushions. Because of the fragile nature of the foam, there is no warranty for the cushions.

**C The warranty period begins on the date you receive the chair. For warranty service, please contact Nuprodx, Inc. no later than one month following the applicable warranty term. The chair will be repaired or replaced at the discretion of Nuprodx, Inc. with no charges to you for parts and labor, provided you have proof of purchase and of purchase date.**

**D Please do not return the chair to Nuprodx, Inc. without the advance authorization of Nuprodx. Freight charges to the factory are at your expense, unless otherwise authorized by Nuprodx, Inc.**

**E DISCLAIMER:** Except for the above warranty, and the acknowledgement by Nuprodx, Inc. that the chair, as manufactured by it, is fit for the general purpose for which most persons acquire a chair of its kind, Nuprodx, Inc. provides that you accept the chair as is, without warranties, either express or implied. Nuprodx, Inc. makes no warranty of fitness for your particular purpose and no warranty of merchantability beyond that already stated. No warranties extend beyond the duration of the express warranty stated above.

**F WARRANTY LIMITATIONS AND EXCLUSIONS:** The only obligation of Nuprodx, Inc. is to provide the purchaser with free repair and replacement as described above. This exclusive warranty remedy will not have failed as long as Nuprodx, Inc. is willing and able to repair or replace as described, but if this remedy should be held to have failed, the only remaining warranty obligation of Nuprodx, Inc. shall be to refund the acts beyond the control of Nuprodx, Inc.

**G This warranty gives you specific legal rights, and you may have other rights that may vary from state to state.**

**H This warranty does not apply to problems arising from normal wear, improper operation, improper maintenance, improper storage or similar disclaimer of implied warranties, and some do not allow limitations on how long an implied warranty may last. Some do not allow exclusion or limitation of incidental or consequential damages. So the above limitations or exclusions may not apply to you.**

## 4 PACKING INSTRUCTIONS

Make sure that you don't pack anything either on top or underneath the seat cushion. It's constructed of closed cell foam which doesn't spring back and once dented will stay that way permanently



**A**


**A** Arrange the four legs (with caster wheels attached) on the foam panel at the bottom of the carrying case



**B**

**B** Place the double sided foam panel on top of the legs



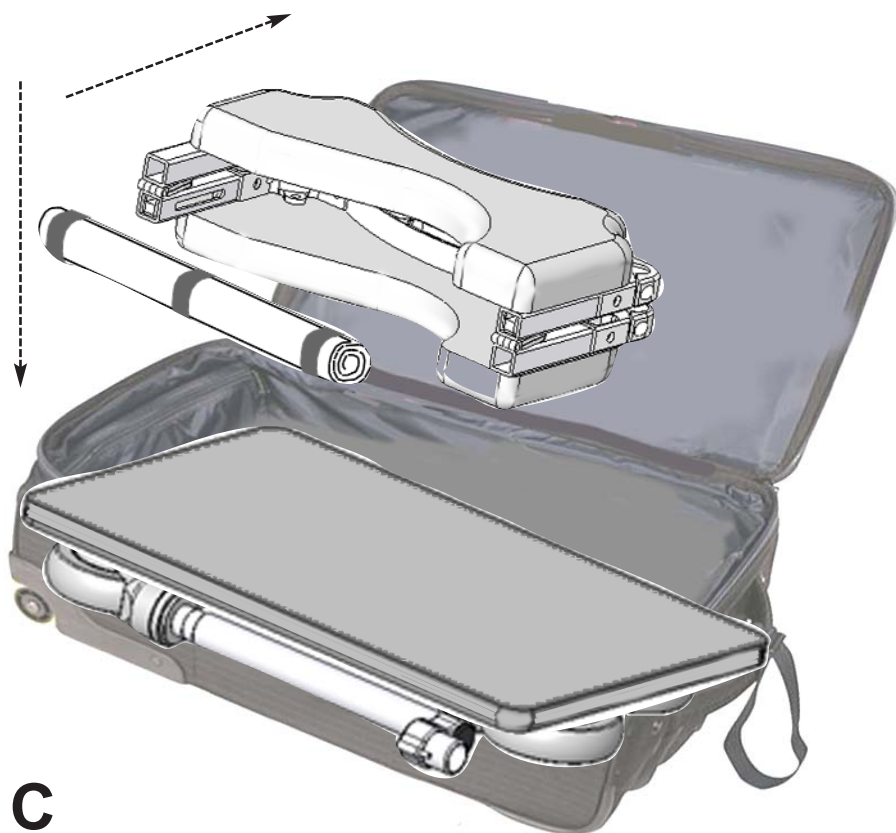

**C** 1) Insert the rolled up cloth back in to the cavity of the seat frame  
2) Place the folded seat frame on top of the double side foam panel

**C**



**D** 1) Place the backrest posts with folded cantilever arms around the seat frame (shown here without the fabric socks)  
2) Secure the legs, seat frame, backrest posts/arms using the black straps which are attached to the carrying case

**D**



**E** Place the folded footrests, tools, & instructions in the zipper pouch located in the lid of the carrying case

**E**



4 Malone Lane, San Rafael, CA 94903  
(415) 472-1699, (415) 492-1396 fax, (888) 288-5653, [info.nuprodx@comcast.net](mailto:info.nuprodx@comcast.net)