


The EquaGel™ Protector Cushion

Advanced Pressure Therapy

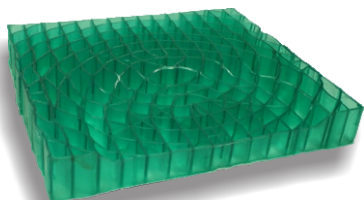


- 100% EquaGel™
- 2 1/2" tall EquaGel™.
- The pelvic area of the cushion is contoured 3/4" deep to more evenly distribute a users weight.
- Fluid resistant, Vapor permeable, flame resistant cover
- Advanced pressure relieving design

Available Sizes:

16"x16"	22"x18"	28"x20"*
16"x18"	22"x20"	30"x16"*
16"x20"	24"x16"*	30"x18"*
18"x16"	24"x18"*	30"x20"
18"x18"	24"x20"	
18"x20"	26"x16"*	*Bariatric
20"x16"	26"x18"*	sizes
20"x18"	26"x20"	have
20"x20"	28"x16"*	foam side
22"x16"	28"x18"*	rails.

The EquaGel™ Adjustable Protector Cushion

Customized Pressure Therapy


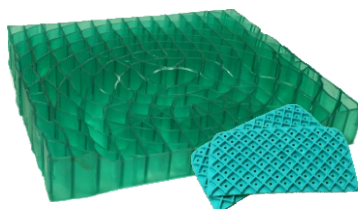


Available Sizes:

16"x16"	22"x18"	28"x20"*
16"x18"	22"x20"	30"x16"*
16"x20"	24"x16"*	30"x18"*
18"x16"	24"x18"*	30"x20"
18"x18"	24"x20"	
18"x20"	26"x16"*	*Bariatric
20"x16"	26"x18"*	sizes
20"x18"	26"x20"	have
20"x20"	28"x16"*	foam side
22"x16"	28"x18"*	rails.

- 100% EquaGel™
- 2 1/2" tall EquaGel™.
- The pelvic area of the cushion is contoured 3/4" deep to more evenly distribute a users weight.
- Fluid resistant, Vapor permeable, flame resistant cover
- Advanced Pressure relieving design
- Included are two different bucket inserts that can change the contour shape and depth of therapeutic gel under the pelvis to: 3/4" deep, 1/2" deep, or 1/4" deep.
- Deeper contour for users with lower pressure needs.

How to Adjust the EquaGel™ Adjustable Protector Cushion:

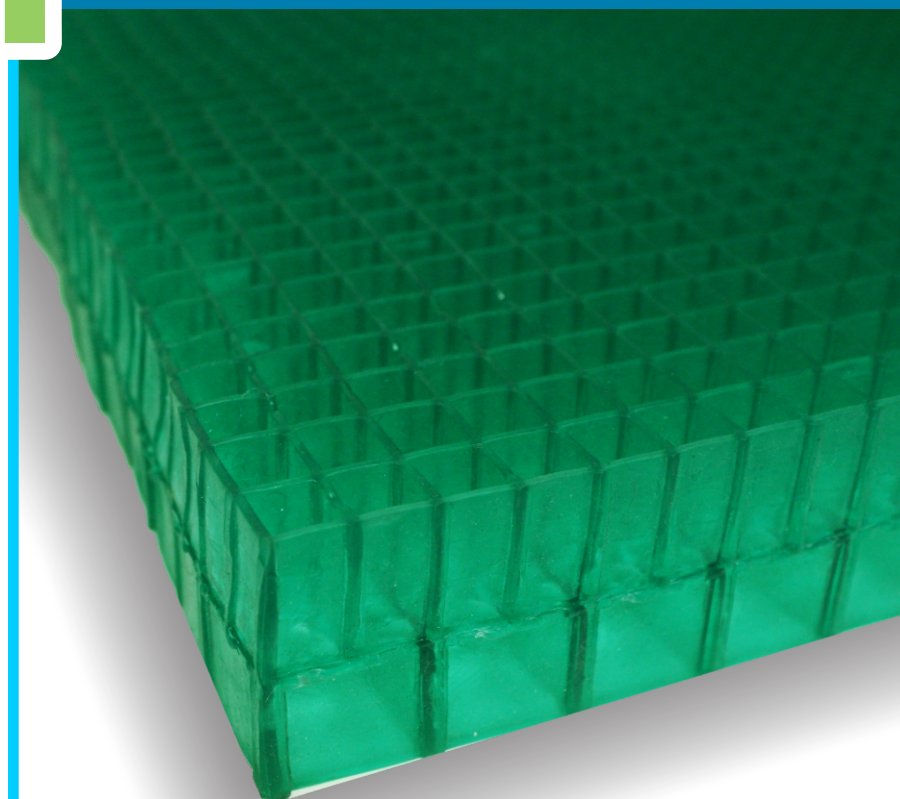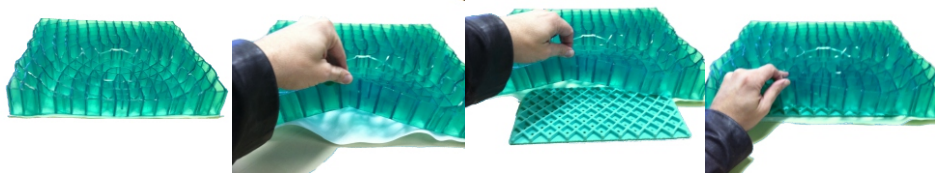


Marketing Concepts LLC

Denver CO 80234

p.303-210-5392 f.303-374-2660

www.Marketingcncpts.com

What is EquaGel™ and How Does it Work?

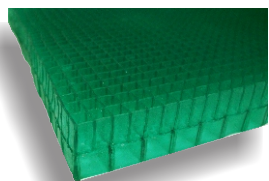

EquaGel™ is a dry-polymer gel, almost rubbery to the touch. It has unbelievable tensile and compression strength and incredible durability; but these are not what make EquaGel™ special. Its most amazing feature is the column buckling.

Simply put, column buckling refers to the way EquaGel™ behaves under weight. Each wall of gel is capable of supporting only a light load. When that threshold is exceeded, the wall of gel buckles into the hollow area between walls and passes the weight to the surrounding walls for support. This “buckling and weight-passing” happens instantly and repeatedly until the maximum surface area of pressing weight is evenly supported by largely equalized pressure (hence our name, EquaPressure).


The peak pressure points of the ischial tuberosities and the coccyx/sacrum buckle into the EquaGel™ feeling almost as if they are free-floating within the cushion while the thighs and buttock of the user are supported.



EquaPressure and EquaGel are trademarks of EdiZONE, LLC of Alpine, Utah USA. EquaPressure, LLC's EquaGel™ cushions are Protected by U.S. Patents 5,749,111, 6,026,527, 6,413,458, 7,060,213, 7,964,664, and, only if the screed molding process is used, 7,666,341.

The EquaGel™ Straight Comfort Cushion


Basic Pressure Therapy

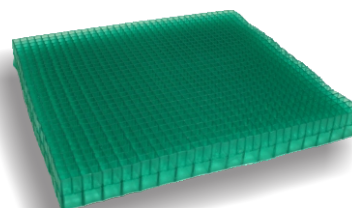

- 100% EquaGel™
- 1 3/4" tall

EquaGel™ in two stages.

- The top layer is thin wall gel and a tight grid for a comfort layer.
- The bottom layer is thicker wall gel and a wide grid for deep posture support.
- Fluid resistant, Vapor permeable, flame resistant cover

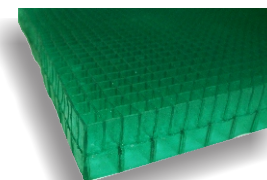


Available Sizes:

16"x16"	22"x18"	28"x20"*
16"x18"	22"x20"	30"x16"*
16"x20"	24"x16"*	30"x18"*
18"x16"	24"x18"*	30"x20"*
18"x18"	24"x20"*	
18"x20"	26"x16"*	*Bariatric
20"x16"	26"x18"*	sizes
20"x18"	26"x20"*	have
20"x20"	28"x16"*	foam
22"x16"	28"x18"*	side rails.

The EquaGel™ General Cushion

Moderate Pressure Therapy

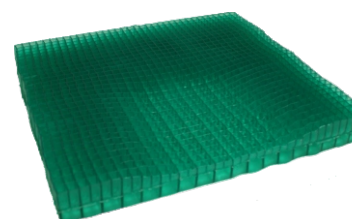

- 2" tall EquaGel™ in two stages.
- The top layer is thin wall gel and a tight grid for a comfort layer.
- The bottom layer is thicker wall gel and a wide grid for deep posture support.
- The pelvic area of the cushion is contoured 3/4" deep to more evenly distribute a users weight.
- Fluid resistant, Vapor permeable, flame resistant cover

Available Sizes:

16"x16"	22"x18"	28"x20"*
16"x18"	22"x20"	30"x16"*
16"x20"	24"x16"*	30"x18"*
18"x16"	24"x18"*	30"x20"*
18"x18"	24"x20"*	
18"x20"	26"x16"*	*Bariatric
20"x16"	26"x18"*	sizes
20"x18"	26"x20"*	have
20"x20"	28"x16"*	foam
22"x16"	28"x18"*	side rails.

EquaPressure cushions are warrantied against defects for 18 months.

While EquaGel cushions are intended to aid the treatment of pressure sores, only a medical professional can determine your treatment needs.